

PHILIPS  
RESPIRONICS

*DreamStation*

Sleep therapy system



# Rediscover dreams



© 2016 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.  
The Bluetooth word mark and logos are registered trademarks owned by  
Bluetooth SIG, Inc. and the use of such marks by Philips Respiromics is  
under license. Wi-Fi is a trademark of Wi-Fi Alliance.  
[www.philips.com/respiromics](http://www.philips.com/respiromics)

Caution: US federal law restricts these devices to  
sale by or on the order of a physician.

RRDPGH SB 11/18/16 MCI 4107797 PN 1132448  
1010 Murry Ridge Lane, Murrysville, PA 15668  
800-345-6443 • 724-387-4000

# Natural. Comfortable. Sleep.

DreamStation supports long-term patient use while creating new efficiencies for your business

#### Powerful patient-driven design

DreamStation was designed with feedback from hundreds of patients to provide a simpler PAP experience to help both new and experienced patients use therapy long-term. A suite of intuitive tools can be customized by healthcare professionals to choose the solution that's right for each patient.

#### Acclimating patients in the early days of therapy

**EZ-Start** can help patients gradually adapt to their prescribed level of therapy using an innovative pressure acclimation algorithm with automatic, personalized adjustments to PAP pressure over the first 30 days.\*\*

**SmartRamp** allows users to fall asleep to lower pressures than standard ramp mode. The air pressure remains low until the patient experiences an obstructive respiratory event. Then SmartRamp responds with an auto algorithm to resolve the patient's apneas.\*\*

#### Make progress every day

Each morning patients are greeted with Daily Progress Feedback — a summary of how far they've come to motivate them to stick with their therapy. The intuitive, color display shows a simple trend of their nightly hours of use, followed by a summary of their last 30 days of good nights with more than four hours of use.

#### Sleep therapy success\* at your patients' fingertips

Bluetooth® is included in every DreamStation device for easy connectivity to the DreamMapper patient self-management tool. DreamMapper helps patients take an active role in their sleep management by providing daily therapy progress, helpful troubleshooting videos, and goal-setting tools. Patients can stay engaged and get the information they need fast, without having to use your staff's time to resolve common issues.

#### An Encore performance

DreamStation connects to our powerful EncoreAnywhere patient compliance management software. EncoreAnywhere is HIPAA compliant and makes it easy and efficient for sleep professionals to manage patient compliance and therapy.

#### Focus your time where it belongs

EncoreAnywhere features Adherence Profiler. It's an optional, intuitive tool that helps you identify patients according to their probability of achieving adherence, helping focus your staff's time on the patients that have the greatest likelihood of success.




\*In a retrospective review conducted by Philips Respironics of approximately 15,000 SystemOne patients, patients who used SleepMapper, which has been rebranded to DreamMapper, demonstrated 22% greater adherence to the therapy than patients who did not use SleepMapper. To see a list of compatible DreamMapper devices, go to [www.dreammapper.com/compatible](http://www.dreammapper.com/compatible).

\*\*Not available in all DreamStation PAP models and modes.



# Creating efficiencies that make a difference

# More cost-effective connectivity options

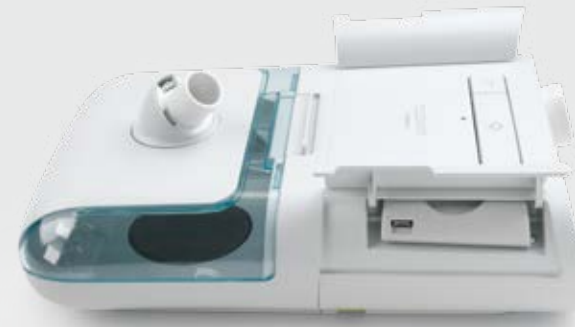

Only DreamStation offers Bluetooth®, Cellular and Wi-Fi® connectivity options. By providing the most comprehensive suite of connectivity choices among leading PAP brands\*, you can choose the options that are most cost effective. DreamStation's innovative design has the capability to adapt as technologies continue to advance.

#### Making it easy to reach your patients

- Cellular provides daily data to clinicians and one hour after the patient has turned the device off
- Wi-Fi delivers more data to clinicians more often and provides coverage even in weak cellular areas
- Bluetooth is included in every device for easy connectivity with DreamMapper and Alice NightOne

\*Internal assessment of 2015 competitive CPAP data comparing to ResMed Aircurve10/Aircurve10 platform and Fisher & Paykel Icon series platform.

## DreamMapper helps patients take an **active role** in their therapy

With the DreamMapper self-management tool, patients can use the mobile and web-based features to engage with their therapy.



## Simplify device evaluation

Performance Check simplifies in-home device evaluation for easy remote troubleshooting. Designed to reduce the time and frustration involved in device troubleshooting, this robust tool allows you to guide the patient through an easy remote diagnosis, resulting in a simple "Pass/Fail" result to determine if the device needs to be returned for service.

Performance Check can also generate a report in our EncoreAnywhere patient management software tool, allowing you to troubleshoot patient concerns.

**According to our research, 25% of returned devices are found to be operating normally.\*** Performance Check is designed to help reduce the number of normally functioning devices that are returned for service, saving you time and delivery costs.

## Reduce manual adjustments

DreamStation can automatically adjust to changing patient needs, helping reduce costly home visits and letting you focus your attention where it needs to be.

- ✓ **CPAP-Check**  
Checks on the user every 30 hours to determine if CPAP pressure is optimal. If not, it automatically adjusts to meet changing patient needs.\*\*
- ✓ **Auto-Trial**  
Delivers breath-by-breath auto-CPAP therapy for up to a total 30 days per patient. Transitions to CPAP or CPAP-Check mode at end of Auto-Trial.\*\*
- ✓ **Opti-Start**  
Provides optimal starting pressure that can enhance patient comfort and reduce the likelihood of residual events at the beginning of therapy.\*\*

\*Philips research, 2015, data on file.  
\*\*Not available in all DreamStation PAP models and modes.

# The performance and appeal patients want

DreamStation offers a sleek, low-profile design. Its vibrant color screen displays daily progress and features easy-to-use menus.

DreamStation's top-mounted humidifier hose swivel and front-facing display make it easy for patients to operate while lying down or sitting up in bed. And, it's small and light, making it easy to pack for travel.

**30%  
smaller**

The DreamStation PAP is 30% smaller than the Airsense10 PAP.\*\*†

**39%  
lighter**

The DreamStation PAP is 39% lighter than the Airsense10 PAP.\*\*†

**63%  
quieter**

The DreamStation PAP is 63% quieter than ResMed Airsense10.\*\*†

Hear it from **DreamStation users**

Good-looking piece of equipment! It's modern looking, doesn't look like a medical machine. It's quieter, too.


I liked it as soon as I opened it! I liked the way it was laid out and I liked the size of it — it was small, easy to fit on a table along with a lamp and my phone.

\*Philips internal comparison, 2015, data on file.

\*\*Based on the average change in sound level during simulated patient breathing, across a range of pressure, expiratory relief, and humidifier conditions.

†Does not apply to non-invasive ventilation products

# Insight-driven innovation

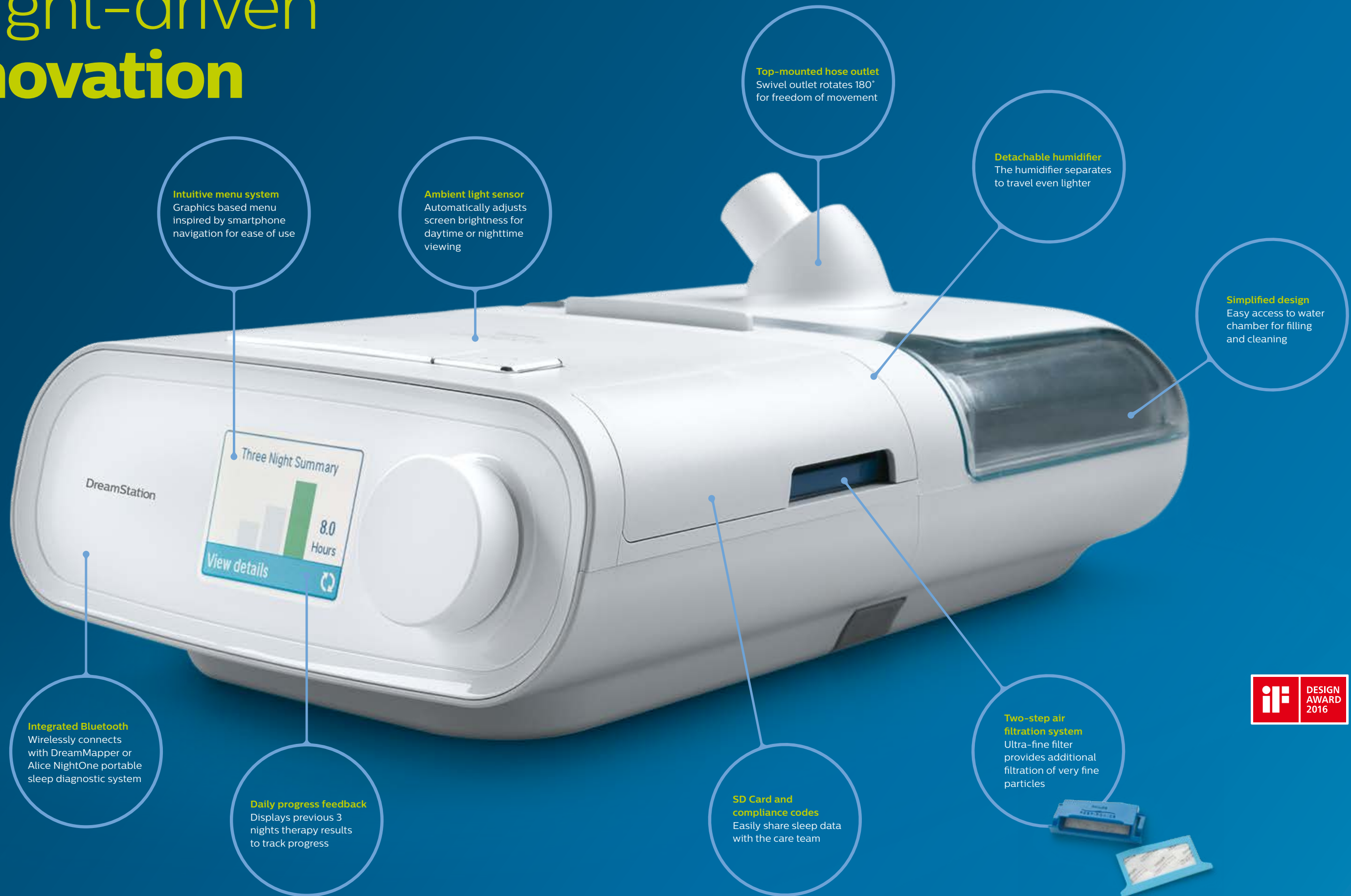

# Ordering information

## APAP/CPAP Sleep therapy

DreamStation Auto CPAP	Part number	With cellular modem	With Wi-Fi modem
DreamStation Auto CPAP	DSX500S11	DSX500S11C05	DSX500S11W05
DreamStation Auto CPAP with humidifier	DSX500H11	DSX500H11C	DSX500H11W
DreamStation Auto CPAP with humidifier and heated tube	DSX500T11	DSX500T11C	DSX500T11W
DreamStation CPAP Pro	Part number	With cellular modem	With Wi-Fi modem
DreamStation CPAP Pro	DSX400S11	DSX400S11C05	DSX400S11W05
DreamStation CPAP Pro with humidifier	DSX400H11	DSX400H11C	DSX400H11W
DreamStation CPAP Pro with humidifier and heated tube	DSX400T11	DSX400T11C	DSX400T11W
DreamStation CPAP	Part number	With cellular modem	With Wi-Fi modem
DreamStation CPAP	DSX200S11	DSX200S11C05	DSX200S11W05
DreamStation CPAP with humidifier	DSX200H11	DSX200H11C05	DSX200H11W05
DreamStation CPAP with humidifier and heated tube	DSX200T11	DSX200T11C05	DSX200T11W05

## Bi-level Sleep therapy

DreamStation Auto BiPAP	Part number	With cellular modem	With Wi-Fi modem
DreamStation Auto BiPAP	DSX700S11	DSX700S11C05	DSX700S11W05
DreamStation Auto BiPAP with humidifier	DSX700H11	DSX700H11C	DSX700H11W
DreamStation Auto BiPAP with humidifier and heated tube	DSX700T11	DSX700T11C	DSX700T11W
DreamStation BiPAP Pro	Part number	With cellular modem	With Wi-Fi modem
DreamStation BiPAP Pro	DSX600S11	DSX600S11C05	DSX600S11W05
DreamStation BiPAP Pro with humidifier	DSX600H11	DSX600H11C	DSX600H11W
DreamStation BiPAP Pro with humidifier and heated tube	DSX600T11	DSX600T11C	DSX600T11W
DreamStation BiPAP autoSV	Part number	With cellular modem	With Wi-Fi modem
DreamStation BiPAP autoSV	DSX900S11	DSX900S11C05	DSX900S11W05
DreamStation BiPAP autoSV with humidifier	DSX900H11	DSX900H11C	DSX900H11W
DreamStation BiPAP autoSV with humidifier and heated tube	DSX900T11	DSX900T11C	DSX900T11W

## Respiratory therapy

DreamStation BiPAP AVAPS	Part number	With cellular modem	With Wi-Fi modem
DreamStation BiPAP AVAPS	DSX1130S11	DSX1130S11C05	DSX1130S11W05
DreamStation BiPAP AVAPS with humidifier	DSX1130H11	DSX1130H11C	DSX1130H11W
DreamStation BiPAP AVAPS with humidifier and heated tube	DSX1130T11	DSX1130T11C	DSX1130T11W
DreamStation BiPAP S/T	Part number	With cellular modem	With Wi-Fi modem
DreamStation BiPAP S/T	DSX1030S11	DSX1030S11C05	DSX1030S11W05
DreamStation BiPAP S/T with humidifier	DSX1030H11	DSX1030H11C	DSX1030H11W
DreamStation BiPAP S/T with humidifier and heated tube	DSX1030T11	DSX1030T11C	DSX1030T11W

## Sleep and Respiratory therapy

Humidification	Part number
DreamStation humidifier	DSXH
DreamStation water chamber	1122520
RP-DS Humidifier Dry Box Assembly	1120668
RP-DS Humidifier Flip Lid Seal	1120617
RP-DS Humidifier Dry Box Inlet Seal	1120613
Tubes	Part number
15mm Std Tube – DreamStation - RP	PR15
15mm Heated Tube	HT15
Filters	Part number
Pollen filter, reusable (1 per pack)	1122446
Ultra-fine filter, disposable (1 per pack)	1122447
Ultra-fine filter, disposable (2 per pack)	1122518
Ultra-fine filter, disposable (6 per pack)	1122519

## Sleep and Respiratory therapy accessories

Power management	Part number
DreamStation 65W power supply*	1120136
DreamStation 80W power supply	1118499
Power cord	1005894
PAP Lithium Ion Battery Kit**	1124009
Li Ion battery, PAP device cable	1121736
Li Ion Battery, Cable Adapter (50 series adapter, 2/each)	1127799
Li Ion Battery Case	1126947

## Sleep and Respiratory therapy accessories

Oximetry	Part number
Link module	1120293
Nonin SpO <sub>2</sub> assembly	1121694
Reuseable finger sensor (clip)	936
Oximetry finger sensor, adult, flex	953
Flexiwrap, sens tape, adult, 25/pk	954A
Reuseable finger sensor (soft), medium	1062050
Reuseable finger sensor (soft), small	1062051
Connectivity	Part number
Cellular modem, oximetry capable	100600C
Cellular modem, Non-oximetry	100605C
Wi-Fi accessory, oximetry capable	100700W
Wi-Fi accessory, Non-oximetry	100705W
Travel	Part number
Universal PAP/laptop travel briefcase	1114784
DreamStation replacement carrying case	1121162
Shielded DC cord	1120746
Shielded 12V DC power cord system (includes shielded DC cord and battery adapter cable)	1120747
DreamStation Travel case, 65W power supply* w/ USB port	1120135
DreamStation Travel case, 65W power supply* w/ USB port, 5/case	1124301
Mask and tubing bedside organizer	Part number
Bedside organizer	1122135
Bedside organizer, 10/case	1124224
Roll stand	Part number
Roll stand	1131859

## Advanced therapies

DreamStation ventilation solutions use clinically proven algorithms that support customized, connected care for patients who require advanced therapy. DreamStation BiPAP autoSV, BiPAP AVAPS, and BiPAP S/T options are designed to offer exceptional comfort, intelligent monitoring, and seamless connectivity between patients and care teams.

Call 1-800-345-6443 or visit [www.philips.com/dreamstation](http://www.philips.com/dreamstation) for more information.

\*Used with flow generator only. Cannot be used with a humidifier.

\*\*Includes a rechargeable lithium ion battery pack, a PAP device cable, and a battery case.

\*\*\*Black face plate only available with DreamStation BiPAP auto SV, BiPAP AVAPS, and BiPAP S/T options.



BiPAP autoSV\*\*\*